

**AUDISOL<sup>®</sup>**  
**DRY EARS**



**FAST ACTING FORMULA**

**PLEASE READ ENCLOSED LEAFLET  
CAREFULLY BEFORE USE. USE ONLY AS  
DIRECTED. IF SYMPTOMS PERSIST  
CONSULT A HEALTHCARE PROFESSIONAL .**

#### *What is AUDISOL<sup>®</sup> Dry Ears?*

AUDISOL<sup>®</sup> Dry Ears is a water removal spray for the ear. Its quick action formula eliminates water and moisture trapped in the external ear canal. Audisol Dry Ears prevents symptoms of Swimmers Ear.

#### *What Causes Swimmer's Ear?*

After sports like swimming, water and moisture can accumulate and get trapped in the ear, sometimes behind an accumulated ear plug in the ear canal. This causes discomfort and if left untreated can lead to the feeling of pressure in the ear, temporary impairment of hearing and possible bacterial growth leading to infections.

#### *How Does AUDISOL<sup>®</sup> Dry Ears Work?*

AUDISOL<sup>®</sup> Dry Ears relieves symptoms by quickly drying excess moisture in the ear canal. The AUDISOL<sup>®</sup> micro spray delivers optimal dispersion in the ear and provides quick relief.

#### *When to Use AUDISOL<sup>®</sup> Dry Ears?*

**For best results, use AUDISOL<sup>®</sup> Dry Ears immediately after swimming, watersports, bathing or showering.**

AUDISOL<sup>®</sup> Dry Ears is indicated for the dispersion and removal of water and moisture in the external ear canal for adults and children over 2 years of age.

#### *Directions for Use*

Spray 1-2 times in the affected ear, trapped water should be quickly released and run out of the ear canal.

**FOR EXTERNAL USE ONLY**

## Warnings

- AUDISOL® Dry Ears should not be used in patients who have a perforated ear drum or inflammation of the ear, are using grommets, suffering from an ear infection or have had recent ear surgery.
- Keep out of the sight and reach of children.
- Do not spray into the eye or mouth. In case of accidental contact with eye, wash thoroughly with running water and consult a medical professional.

## Composition

Isopropyl Alcohol 93 %, Glycerin 4.995 %, Acetic Acid 2 %, Rose oil 0.005 %.

## Storage Conditions

Store in a cool place.

Do not expose to temperature over 50°C.

ARTG No : 291711

## Distributed By

UK	Noru Pharma PVT Ltd, Medicity - D6, Thane Road, Notingham NG90 6BH
AUSTRALIA	Noru Pharma, Unit 10, 50 Victoria Road, Drummoyne, NSW 2047
NEW ZEALAND	Noru Pharma Ltd, 48B Arundel Street Hillsborough Auckland 1041
HONG KONG	Noru Pharma (H.K.) Ltd, Flat U, 7/F, Valiant Industrial Centre, 2-12 Au Pui Wan Street, Fotan, NT
SINGAPORE	Noru Pharma Pte. Ltd, 1, Ang Mo Kio Industrial park 2A # 04-03 AMK Tech 1, Singapore- 568049

## Product Range Details:

Also available from AUDISOL®:

- AUDISOL® Ear Wax Remover
- AUDISOL® Ear Cleansing Spray
- AUDISOL® Pain Relief Spray

## Meaning of Symbols Used



The CE marking is the manufacturer's declaration that the product meets the requirements of the applicable European Directive.



Symbol indicating that the product packaging is able to be recycled.



Manufacturer information.



The product should not be used after the expiry date.



Batch number of the product.

A product of  **Noru Pharma**  
Live Better

**More Information:** [norupharma.com](http://norupharma.com)

 **Manufactured by:**  
**Danima Smart Solutions B.V**

Herengracht 74, 1015 BR Amsterdam  
The Netherlands

AUDISOL® is a registered trademark.

This Instruction For Use was last revised 20 July 2017.