

AUDISOL® EAR CLEANSING SPRAY



**Better alternative
to Cotton Buds**

- Washes away ear wax
- Prevents build up of ear wax
- Keeps ears feeling clean and fresh

**READ THIS LEAFLET CAREFULLY BEFORE USE.
USE ONLY AS DIRECTED.**

**IF SYMPTOMS PERSIST CONSULT A
HEALTHCARE PROFESSIONAL.**

SUITABLE FOR CHILDREN FROM THE AGE OF 6.

What is Audisol® Ear Cleansing Spray?

Audisol® Ear Cleansing Spray is a gentle, effective, natural and non-medicated spray which helps in cleaning your ears of wax build up. Audisol® does not contain any chemicals or preservatives.

What causes ear wax build up?

Ear wax is a natural production of the ear. However, excessive production of ear wax by the body and usage of cotton buds can push ear wax deeper into your ears and cause wax build up, and in some cases, ear plugs.

Directions for Use

- Ensure Audisol® spray is used at room temperature (15 - 25°C) and that the transparent end-piece is securely fitted over the nozzle before and during use.
- Insert the nozzle as far as the nozzle allows you to.
- Give one short squirt in one or both ears as needed;
 - ◆ usually 2 to 3 times per week.
 - ◆ or 1-2 times a day when needed, for example when using hearing aids or earplugs frequently or when experiencing symptoms of wax related obstruction in the ear canal.
- Tip the head to the side and wipe away any residual liquid in the outer ear. Do not dry very vigorously as this may cause inflammation of the sensitive skin in the ear.
- Rinse the transparent end-piece of the nozzle with warm soapy water after each use and leave to dry.

Warning

Pressurised container: May burst if heated.

- Do not pierce or burn, even after use.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- Protect from sunlight.
- Do not expose to temperatures exceeding 50°C/122°F.
- Keep out of reach of children.

Precautions

- The use of the same can of Audisol® by more than one person may spread infections, so clean nozzle in warm soapy water and rinse after each use.
- Do not use after the expiry date.

Important Information

Do not use Audisol® ear cleansing spray if

- You are allergic or hypersensitive to any of the ingredients.
- You have a history of perforation of the ear drum.
- You have had ear surgery.
- You have pain in your ears or your ears are inflamed or have an infection.
- There are any foreign bodies present in the ear canal.
- You are currently using other ear drops, of any kind.
- You are currently being treated with a tympanic drain (Grommets) or Pressure equalizing tubes (PETs) in your ears.

Do not use in children below the age of 6 years.

Composition

Natural seawater (diluted to Isotonic physiological level (0.9%) with Purified Water).

DISTRIBUTED BY

UK	Noru Pharma Ltd, Medicity - D6, Thane Road, Nottingham NG90 6BH
AUSTRALIA	Noru Pharma, Unit 10, 50 Victoria Road, Drummoyne, NSW 2047
NEW ZEALAND	Noru Pharma Ltd, 48B Arundel Street Hillsborough Auckland 1041
HONG KONG	Noru Pharma (H.K.) Ltd, Flat U, 7/F, Valiant Industrial Centre, 2-12 Au Pui Wan Street, Fotan, NT
SINGAPORE	Noru Pharma Pte. Ltd, 1, Ang Mo Kio Industrial park 2A # 04-03 AMK Tech 1, Singapore- 568049

ARTG No : 277399

CE 0413

Manufacturer:

Aurena Laboratories AB

Fjärrviksvägen 22 SE-653 50 Karlstad, Sweden

A product of Noru Pharma
Live Better

More Information: norupharma.com

AUDISOL® is a registered trademark.

REF 1013-3 STERILE R

50ml

LOT See bottom of can/box.

Version 20170919

MEANING OF SYMBOLS USED

REF ¹⁰¹³	Catalogue number of the product.	15/25	The upper and the lower temperature limits for the product. The product should be stored at room temperature (15-25°C).	CE xxxx	CE marking with the four digit identification number of the notified body involved in the conformity assessment procedure. The CE marking is the manufacturer's declaration that the product meets the requirements of the applicable European Directive.
STERILE R	Sterilization symbol indicating sterilization by irradiation.		The reversed epsilon indicates that the aerosol dispenser satisfies the requirements of the Aerosol Dispensers Directive.		
	Manufacturer information.		The product should not be used after the expiry date.		
LOT	Batch number of the product.		Store this product out of sight and reach of children.		Symbol indicating that the product packaging is able to be recycled.